



# **ELECTRODE PLACEMENT AND CARE**

## **KNEE PLACEMENT**

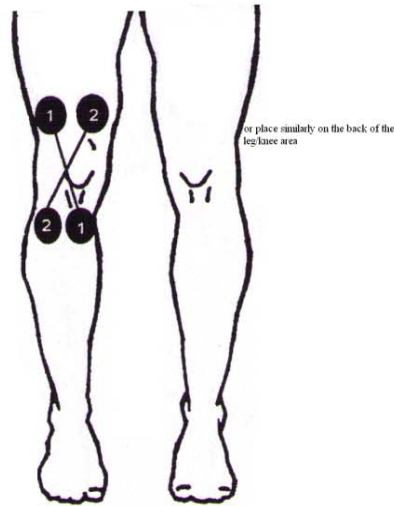

Primary Placement

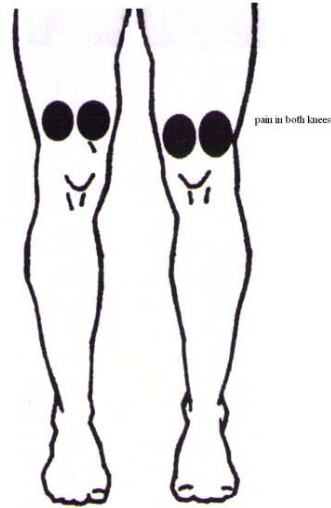

Alternate Placement

### **WARNING:**

- \*Do not place electrodes on the side or front of neck.
- \*Keep electrodes away from carotid artery.
- \*Do not overlap electrodes.
- \*Do not place on spine.
- \*Do not immerse in water.

### **Electrode Care:**

**First use:** Attach lead wire to electrode; reserve plastic back and zip-lock bag. Attach electrode to clean dry skin. Turn unit on for treatment as instructed.

**After treatment:** Carefully peel electrode from skin and place back on plastic sheet. Carefully disconnect lead wires from electrodes. Place electrodes in zip-lock bag and store in refrigerator.

**To reuse:** Remove zip-lock from refrigerator and remove electrodes from bag. Carefully attach lead wires. Work with 1 pair of pads at a time. Gently peel from plastic back. Using index finger, add a drop of water to gel on the back of the electrode; smear gently and wait 15 seconds. Re-apply the electrode to clean dry skin. Repeat for remaining pair of electrodes.